

Multi-Gyn® FloraPlus

Multi-Gyn®

PLEASE READ THIS INFORMATION LEAFLET CAREFULLY BEFORE USING MULTI-GYN FLORAPLUS AND KEEP IT AT HAND FOR REFERENCE.

Multi-Gyn FloraPlus relieves vaginal yeast symptoms

- Treats symptoms of vaginal yeast
- Relieves burning, itching, irritation
- Reduces white, crumbly discharge
- With prebiotics and lactic acid
- Optimal pH for a healthy vagina
- Based on plant derived ingredients

Indication

Experiencing symptoms of vaginal yeast? Don't despair – it is a common condition and Multi-Gyn FloraPlus will help you cope with the symptoms in a fast and effective way. Multi-Gyn FloraPlus effectively treats vaginal yeast symptoms and related discomforts, such as burning, itching, irritation, and white, crumbly discharge. Multi-Gyn FloraPlus is formulated with natural 2QR-complex, lactic acid and prebiotics. Together, they create a protective physical barrier over vaginal mucosa to prevent contact with harmful bacteria and yeast. They also support the 'good lactobacilli' naturally occurring in the vagina and support a healthy flora. The gel has an optimal pH for a healthy vaginal flora and is easy to use thanks to single dose tubes. Free from hormones, fragrance, and animal derived ingredients. Multi-Gyn FloraPlus will bring you back to brilliant in no time.

How Multi-Gyn FloraPlus works

Multi-Gyn FloraPlus is a gel containing 2QR-complex, a plant based ingredient. The gel creates a physical protective barrier over the vaginal mucosa to ensure a healthy vaginal flora. This barrier prevents harmful bacteria and yeast from contacting the vaginal tissue and mucosa.

Content of pack

5 single dose tubes of 5 ml each + instructions for use

Directions for use

To relieve vaginal yeast symptoms, use 1 single dose tube before bedtime for 5 consecutive days.

Multi-Gyn FloraPlus should not be used for longer than 30 consecutive days. This allows the vaginal flora to regulate itself without support.

Warning

If symptoms persist or recur, talk to your healthcare professional.

Multi-Gyn FloraPlus is based on plant derived ingredients

Multi-Gyn FloraPlus is formulated with 2QR-complex in combination with prebiotics. This unique formulation is free from any hormones or fragrance.

Ingredients

2QR-complex* (60%), Sodium Benzoate, Citric acid, Potassium Sorbate, Aqua, Lactic Acid, Glycerin, Sodium Levulinate, Sodium Anisate, Sodium Hydroxide, Xanthan Gum, Fructo-oligosaccharides (FOS), Lactitol, Hydroxypropyl Methylcellulose, Caprylyl Glycol.

*Purified from Aloe Barbadensis Leaf Extract

Allergy information

Allergies to the ingredients of Multi-Gyn FloraPlus are very rare. Please refrain from using Multi-Gyn FloraPlus if you are allergic to any of the ingredients.

Please note

If you feel a burning and/or tingling sensation after application – don't worry. This is normal and passes quickly.

Side effects and Contraindications

Multi-Gyn FloraPlus has an optimal acidity (low pH) for the vagina and may reduce the activity of sperm, which is alkaline (high pH). Therefore we recommend not to use Multi-Gyn FloraPlus during your ovulation period if you wish to become pregnant.

Keep in mind that this product does not treat or prevent Sexually Transmitted Diseases.

Interference with medication

Multi-Gyn FloraPlus has no known interactions with medication.

Pregnancy and lactation

There are no known contraindications for the use of Multi-Gyn FloraPlus during pregnancy and while breastfeeding. Discuss the options with your healthcare professional.

Use with condoms and pessaries

The effect of Multi-Gyn FloraPlus on a pessary and condoms is unknown.

Use with tampons and vaginal rinses


It is not recommended to use Multi-Gyn FloraPlus concomitantly with tampons and vaginal rinses. They may remove (part of) the Multi-Gyn FloraPlus from the vagina and therefore reduce the effect of the product.

Storage


Keep Multi-Gyn FloraPlus at a temperature between 10°C and 25°C (room temperature). Store Multi-Gyn FloraPlus out of reach of children. Do not use Multi-Gyn FloraPlus after the expiry date.

How to use Multi-Gyn FloraPlus


Multi-Gyn FloraPlus is a vaginal gel which is applied directly into the vagina. Each tube is a single dose application. The drawings below show how to use the single dose tube. Dispose of the tube after use, for hygienic reasons.

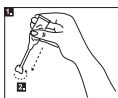


1. Shake the tube contents towards the tip of the tube.
2. Break off the tip of the tube by twisting it.

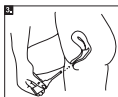

3. Gently insert the thin end of the applicator (of the tube) into the vagina.

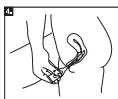

4. Gently squeeze the tube to apply the gel into the vagina.

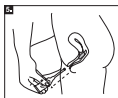

5. Continue squeezing the tube while removing it.

Note: The tube is designed to apply the recommended amount of gel. When used as directed, a small amount of gel may remain in the tube.



Distributed by:

NP Brands Pty Ltd, 120 Woodlands Drive, Braeside, VIC 3195, Australia
NP Brands (NZ) Ltd, PO Box 37217 Parnell, Auckland 1151, New Zealand



Manufactured by: BioClin BV, Blaak 555,
3011 GB Rotterdam, The Netherlands

